

Procedural Oxygen Mask™

POMELITE™ EZ SERIES

MEDIUM CONCENTRATION



1. Choose oral or nasal membrane port
Push finger from inside mask through the EZ tear membrane.
Fully separate connecting edges within membrane.



PRE-ASSEMBLED

- O₂ line attached to medium concentration adapter



3. First, unscrew blue cap. Then, securely attach [Leur-Lok] EtCO₂ sample line to threaded EtCO₂ adapter port on mask.

- Make sure O₂ and EtCO₂ sample lines are functioning properly.



POMELITE LT™ SERIES


HIGH CONCENTRATION



1. Choose oral or nasal membrane port
Push finger from inside mask through the EZ tear membrane.
Fully separate connecting edges within membrane.



- Insert Medium or High Concentration non-rebreather adapter into O₂ port on mask.
- Make sure non-rebreather adapter is fully seated and flush with mask.
- Make sure one-way valve or non-rebreather adapter is working properly.

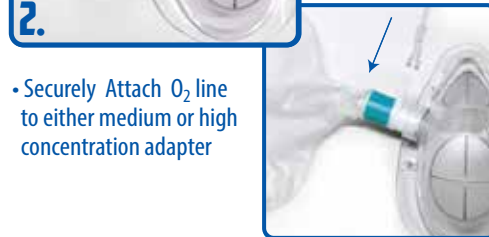

- Securely Attach O₂ line to either medium or high concentration adapter



3. First, unscrew blue cap. Then, securely attach [Leur-Lok] EtCO₂ sample line to threaded EtCO₂ adapter port on mask.

- Make sure O₂ and EtCO₂ sample lines are functioning properly.

